

## HANDRAIL END LOOPS

PART NO. MLPPC

TOOLS REQUIRED: 5/32 hex wrench (supplied)

1. Install Handrail End Loops as shown below. Loops are installed correctly when
  - a. loops at top of ramp are parallel to the walking surface of the platform/porch/landing area for 12", and
  - b. loops at bottom of ramp are parallel to the walking surface of the platform/landing ground area for 12".
2. Using supplied hex wrench, ensure that the Ring Joiners are secured firmly and tightly to the handrails and loops.

