

Press PWR to turn On or Off.
15 Minute Timer: Single Press (GREEN)
30 Minute Timer: Press & Hold (ORANGE)

Press LO or HI to turn HEAT On or Off.
Independent 30 Minute Timer.

Press any or all ZONE keys to turn any
Zone On or Off.

Press SELECT, WAVE or PULSE for
desired operating mode.

Press - (Left) to decrease or + (Right) to
increase INTENSITY and SPEED.



GOLDEN
technologies

www.goldentech.com

Select Your Ideal Comfort Level with Our Unique Features!

Dozens of programmable massage features using six massage motors provide relief to stiff, tired and strained muscles.



Program any or all of the three massage zones while in Select, Wave or Pulse massage settings for personalized comfort.



Soothing LO and HI penetrating lumbar heat, along with an independent 30 minute heat timer.



Backlit hand control is easy to read and understand - even in low light!



Choose from the 15 minute massage timer (**GREEN**) or the 30 minute massage timer (**ORANGE**) - fall asleep to the relaxing massage vibration.



This product is low voltage for safe usage.

LIMITED MANUFACTURER'S WARRANTY

© COPYRIGHT 2008 INSEAT SOLUTIONS, LLC
SANTA FE SPRINGS, CA