

BEDSIDE VALET - ASSEMBLY INSTRUCTIONS

When properly attached to a metal bed frame, per the instructions below, this product is intended to assist a person in sitting up from a prone position in bed. It is also intended to help a person stand up or sit down while getting into and out of bed.

►► WARNINGS ◀◀

- This product is not intended to support full body weight.**
- Do not use as a ladder or stand on crossbars.**
- Do not use as a walker.**

Parts List

1.	59009	Bedside Valet - Opal
2.	59018	Bottom Clamp - Zinc
3.	AG01	Adjustable Floor Glide
4.	CG01	Carpet Gripper
5.	TK01	Clamping Knobs

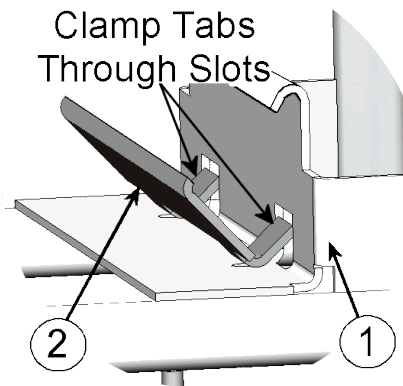


Fig. 1

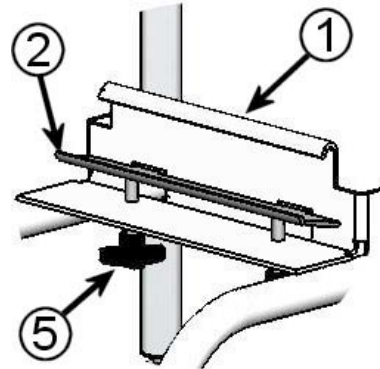


Fig. 2

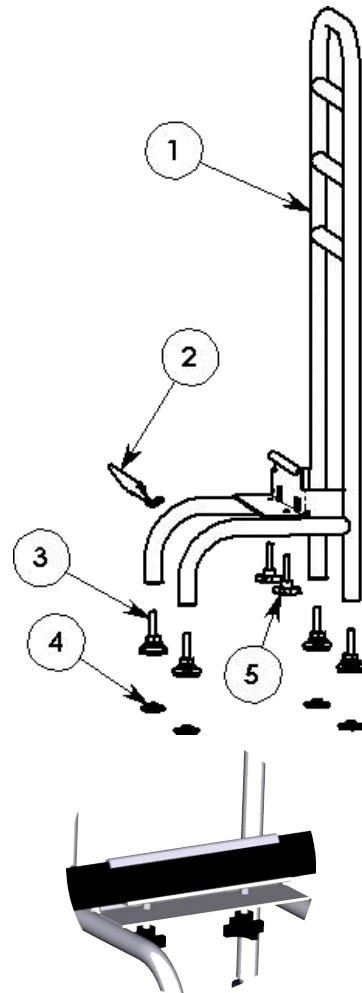


Fig. 3

ASSEMBLY INSTRUCTIONS

- 1 Remove Bedside Valet (1) from the shipping container and cut the plastic strap retaining the hardware bag. Thread the leveling glides (3) into the bottom holes. If the unit will be used in a room with carpeting, snap the carpet grippers (4) onto the leveling glides.
- 2 Make sure the bottom clamp (2) is assembled with the tabs inserted into the slots of the Bedside Valet, with the tabs facing up Fig. 1
- 3 Unthread the two clamping knobs (5) from the bottom, just far enough to keep them from falling out. Fig. 2
- 4 Attach the assembled Bedside Valet to the side of the metal bed frame, approximately 24 inches from the headboard. Select a location within a comfortable reach for the person who will be using Bedside Valet. Slip the rolled-over lip on the vertical plate (1) over the bed frame and tighten the clamping knobs (5) to press the bottom clamp (2) against the bottom of the frame. Make certain that the bottom clamp fits over the inner edge of the bed frame and the frame is securely locked in position. Fig. 3
- 5 Adjust the leveling glides down to contact the floor. Adjust all four glides evenly, using the 1/2" open-end wrench (provided) to turn the adjusting nut.