3053 WALK-IN BATHTUB 30" x 53"

Key Attributes

Walk-in Type:	Walk-in
Material:	Gel Coat
Soaker Weight:	120 lbs
Jetted Weight:	155 lbs
Average Fill Capacity*:	50-60 ga

*Estimated with occupant inside. May vary based on occupant's volume.

Required Electrical	Service 110v
Air Bath or Whirlpool	1x 15 amp GFCI
Whirlpool w/ Inline Heater	1x 20 amp GFCI
Whirlpool & Air Jetted	1x 20 amp GFCI
Soaker	n/a


Available Therapy Systems

3053**S	Non-Jetted Soaking
3053**H	Whirlpool Jetted
3053**A	Air Jetted
3053**D	Whirlpool & Air Jetted

Key Measurements (inches)

, , , , , , , , , , , , , , , , , , , ,	,
Overall Outer:	52.375 x 29.875 x 39
Interior Width:	23.75
Standing Area:	20 x 19.5
Seat Width:	21
Seat Height:	17
Soaking Depth:	32
Drain to Side Wall:	10
Drain to Back Wall:	14.5
Step-in Height:	7
Door Clearance:	13
Faucet Deck:	4.5
Tile Flange:	1
Extension Kit:	7

It is vital that tub is completely level with all leveling feet touching the floor for the door, seal and drain to work properly.

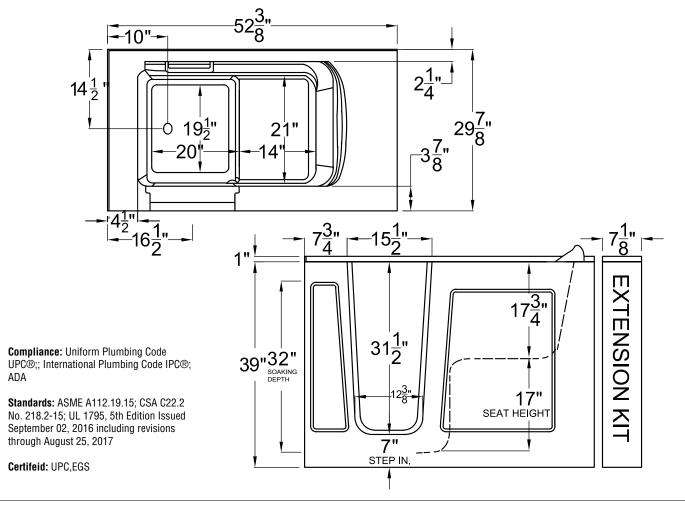

- IMPORTANT! TEST THE TUB'S DRAIN AND JETS (if any) BEFORE PERMANENT INSTALLATION!
- INSPECT THE TUB WHEN RECEIVED! Report any damages or missing parts within 5 business days of delivery date.
- Before ordering or installing, please consider future access to equipment.
- · Please measure tub on-site before building tub surrounding!
- Jet, pump & blower locations are predetermined and cannot be changed.
- Tub door must remain closed when not in use.

- Please note that installer(s) must be certified or warranty will be voided.
- Measurements are subject to a +/- .5" variance and subject to change without notice. Measurements here in supersede all other published specs prior to publication date shown below.
- Inline heater (if included) is designed to maintain water temperature during operation. It will not heat cold water to desired bathing temperature.
- Product must be registered for warranty activation. See user manual or visit SWCORP.com/register for more information.